


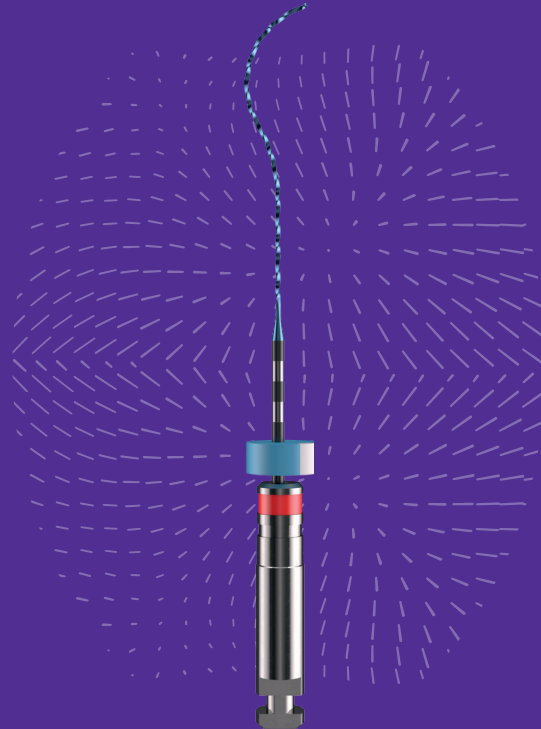

Finisher

Up to
98%
bacterial reduction.*
The universal mechanical
cleaning solution.



www.fkg.ch/finisher


Enhanced cleaning

*Azim AA, Akseil H, Zhuang T, Mashtare T, Babu JP, Huang GT. Efficacy of 4 Irrigation Protocols in Killing Bacteria Colonized in Dentinal Tubules Examined-by a Novel Confocal Laser Scanning-Microscope Analysis.-J Endod. 2016;42(6):928934. doi:10.1016/j.joen.2016.03.009

Finisher



Universally Compatible

Finisher is compatible with all rotary or reciprocating endo systems.

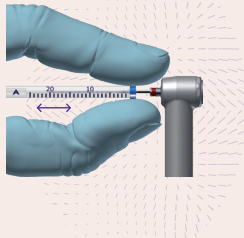

Finisher
25/.00

800-1000RPM – 1Ncm

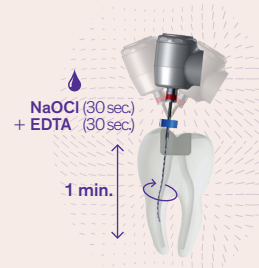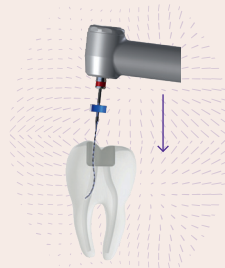

EN

- Shaping min. \varnothing 25
- Set the working length by using the plastic tube

FR

- Mise en forme min. \varnothing 25
- Reporter la longueur de travail à l'aide du tube en plastique

DE

- Aufbereitung min. \varnothing 25
- Die Arbeitslänge mit Hilfe des Kunststoffröhrchens einstellen

- Start with the largest canal
- Insert the instrument pointing towards the cusp and start the motor

- Débuter par le canal le plus large.
- Insérer l'instrument en l'orientant contre la cuspidé et démarrer le moteur

- Mit dem weitesten Kanal beginnen
- Das Instrument Richtung Höcker einführen und den Motor starten

- Clean the entire canal for 1 minute with long and gentle strokes (7-8 mm)
- Proceed to final rinse

- Nettoyer l'intégralité du canal pendant 1 minute à l'aide de longs et doux va-et-vient (7-8 mm)
- Procéder au rinçage final

- Den Kanal für 1 Minute mit langen und sanften Bewegungen (7-8 mm) reinigen
- Abschlusspülung durchführen

Cleaning achieved

