

Your Go-To Temporary Cement

Stays Put, Removes Easily



NexTemp® Non-Eugenol Temporary Resin Cement

NexTemp® is the reliable temporary cement you can count on. That's because NexTemp creates an excellent marginal seal with firm retention to help prevent premature dislodgment or microleakage.

What's more, NexTemp is the only resin cement containing potassium nitrate, chlorhexidine and fluoride – which studies show help mitigate post-op sensitivity.¹



Temporary restorations stay put
Yet can be easily removed



Eugenol-free
No interference with definitive resin cementation



Proprietary formula
Studies show ingredients help prevent sensitivity²



Affordably priced
Good for your bottom line


Firm retention, easy retrieval, exceptional value – Next time it's NexTemp!



premier® Inspired solutions
for daily dentistry®

Premier® Dental Products Company • 888-670-6100 • premierdentalco.com


NexTemp® is easy to use.

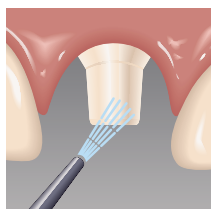


Dry and isolate the prepared tooth. Do not desiccate.




Bleed the syringe. Mount a new automix dispensing tip and bleed the syringe again.

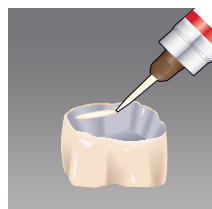


Dispense a **small bead** of cement around the **inside margin**.

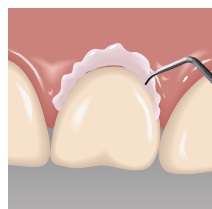

Seat the temporary and hold in place. In approximately 2 minutes, excess cement reaches a distinct gel phase and can be removed safely.

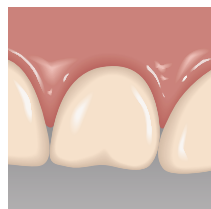

Final set occurs between 4-5 minutes after seating of temporary restoration.*

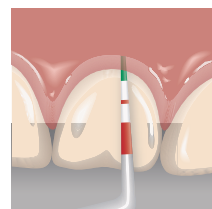

Use a PerioWise® Periodontal Probe for post-treatment periodontal evaluation.

* Final set time is determined intraorally. Ambient set times vary and are not representative of clinical conditions.

Ordering Information

3001471 NexTemp Standard Kit - Shade: Opaque
Contains: One 5ml Automix Syringe, 10 Mix Tips

3001470 NexTemp Bulk Kit - Shade: Opaque
Contains: Four 5ml Automix Syringes

3001451 Syringe Tips
Contains: 25 Automix Syringe Tips

3001472 NexTemp Standard Kit - Shade: Clear
Contains: One 5ml Automix Syringe, 10 Mix Tips

3001473 NexTemp Bulk Kit - Shade: Clear
Contains: Four 5ml Automix Syringes

NexTemp® Clear is ideal for temporary cementation in the aesthetic zone. It gives you the same reliable NexTemp performance but without the pigment.

Flashing Crown Remover



Flashing Crown Remover • 1003576

As recommended by Dr. Donald Flashing



Flashing Crown Remover

Ideal for removing provisionally cemented crowns without damaging margins and is less traumatic to the patient.

premier® Inspired solutions for daily dentistry®



Premier® Dental Products Company • 1710 Romano Drive, Plymouth Meeting, PA 19462, USA
Phone 888-670-6100 • 610-239-6000 • Fax 610-239-6171 • premierdentalco.com • info@premierdentalco.com

1. Jalalian E, Meraji N, Miraei M. A comparison of the efficacy of potassium nitrate and Gluma desensitizer in the reduction of hypersensitivity in teeth with full-crown preparations. J Contemp Dent Pract. 2009 Jan 1;10(1):66-73. / 2. Data on file. / ©2023 Premier Dental Co. | All Rights Reserved